

# SEDI MAS

## Micro erythrocyte sedimentation rate system for single use

### PIPETTE "SEDIMAS" with test-tube and stopper



### PIPETTING

Prepare the blood sample according to the instructions. Carefully mix the prepared sample. Pierce the yellow stopper with SEDIMAS pipette and insert the pipette straight to the bottom of the plastic test-tube; the blood column will automatically rise to its zero position, important for a successful measurement. Upon completion of the pipetting, the system VACUPETA is remains closed, reducing the risk of infection.

### A STANDARDIZED MEASURING PROCEDURE

Insert the pipette with the test tube into a base. Take care that the pipettes stand vertically. The form of the pipette and sample preparation in the test tube ensures a full compatibility with the Westergreen procedure. Read the test results after a period of 1 hour. Upon completion of the test don't disassemble the system; dispose the pipette together with the test tube in accordance with the laboratory regulations.

### BASE FOR 5 PIPETTES



ref	product	note
3.01.1.6	SEDIMAS kit	includes autozeriong pipette, test-tube and stopper
3.01.9.1	SEDIMAS base	base for 5 pipettes